Prof. Bora Korkut

5431-Finishing & Preparation Set

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“The quality of restorative instruments determines the quality of restorations.”

These new kits from BUSCH were compiled by Assoc. Prof. Bora Korkut, DDS, PhD, Faculty of Dentistry, Marmara University Istanbul, Turkey under the guidance of the scientific and clinical experience of many years.

His idea was to offer dentists a professionally curated practical solution.

Each kit includes very high-quality, unique instruments that were carefully selected to meet this demand.

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
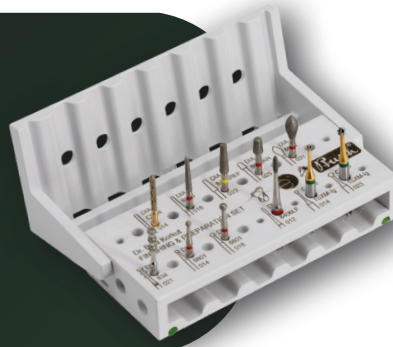
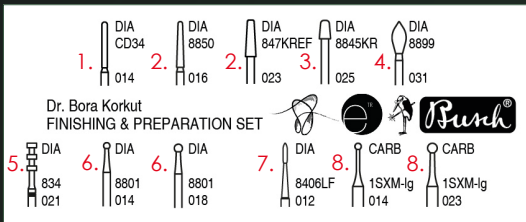
The two sets contain the instruments recommended by Prof. Bora Korkut for restorations.

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1. Preparation for direct and indirect restorations in the anterior and posterior regions. Creation of the secondary and tertiary morphologies of composite restorations.
2. Preparations for direct and indirect anterior veneer restorations. Beveling and levelling of the vestibular surface in composite restorations in the anterior region. Preparation for direct and indirect restorations in the posterior region (inlay/onlay/overlay/crown).
3. Preparation and cavity wall modifications, particularly for indirect restorations in the posterior region such as inlays, onlays and overlays.
4. Cavity wall modifications for direct and indirect restorations. Removal of excess composite in the cingulum region of the anterior teeth for composite restorations.
5. For direct and indirect veneer preparations. With 1.0 mm depth marking.
6. Preparations for direct and indirect restorations in the anterior and posterior regions. Removal of small and large enamel caries lesions.
7. Creation of secondary and tertiary morphologies on composite restorations in the anterior region.
8. Rapid and low-vibration removal of dentine caries.

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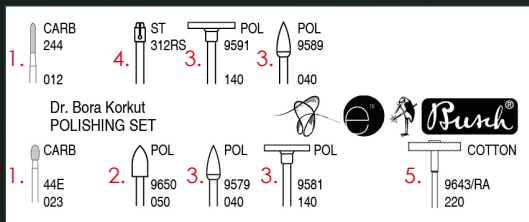1. The 2 multiblade carbide burs can be used to shape the surface of a partial composite restoration while avoiding iatrogenic enamel damage.
2. Unique polisher for resin remnant removal procedure after orthodontic treatment. Together with the carbide burs, a clinician can perform a very safe and elective removal of composite remnants without enamel damage.
3. In addition to the gold standard, two-step, diamond-impregnated, spiral polishing wheels, this kit also has the same two-step polishing burs in flame shape. Therefore, a clinician can either perform anterior or posterior composite restorations surface polishing electively, even the deepest points, by preserving the generated morphology. The flame shape polishing burs can also be used to shape the secondary morphology of anterior composite restorations.
4. This mandrel can be used with any brand of polishing discs (e.g. BUSCH SOFT DISC), to shape a restoration's primary morphology.
5. Cotton buff that can be used together with an extra-fine polishing paste to create the high-end surface gloss and smoothness.