Busch®

There is no substitute for quality

Diamantschleifer Diamond instruments

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The diamond grinders **TOP Grip**, **SIDE Grip** and **KR-Version** are products with passive safety.

All instruments have intelligent, specific properties that have been specially developed for high-risk patients and are indispensable for gentle treatment.

Especially for diabetics, it is a primary goal to perform any treatment injury-free.

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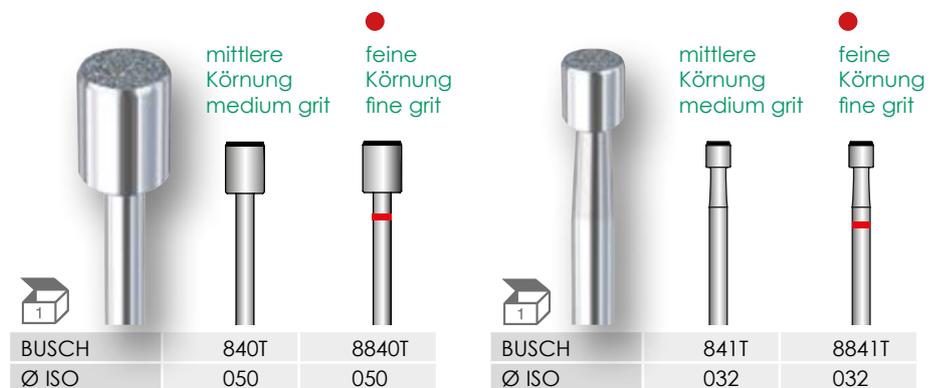
By changes on infected nails the individual TOP GRIP form can shorten the overlying nail edges safer or deformed nail plates can be selectively reduced and adapted accordingly.

Besides the approved cylindrical basic shape 840T we offer the small-sized shape 841T with the option to do filigree treatments. With the shape 828T with diamond coated TOP-Surface unnaturally deformed nails and the overlying nail in front and side areas can be reduced with a better overview and the concave safe smooth side surface fits on front and on sides of the skin contour. The shape 805T with a straight smooth surface lies supporting on the nail wall while reducing nail surface on the sides of the nails.

Basically these filigree shapes offer pedicurists/podiatrists more room to operate.

Generally we recommend a decrease of rotation in the dry technique and also by treatments on high-risk patients.

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- diamond-coated Side surface
- safety bevel
- e.g. flat removal of callus or nails

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Foremost by changes on infected nails and on certain skin parts the individual SIDE GRIP shape can pinpoint individually the areas to be treated or rather to be abraded. Unintentional sliding contacts are avoided by the smooth frontal area.

We offer besides the approved cylindrical and conical basic shape also the small dimensioned cylindrical shape 841S with the option to do filigree treatments. The diamond coated conical SIDE-surface of the shape 805S offers the advantage e.g. to abrade cornification safer in the front and on the sides of the nail edges.

Generally we recommend a decrease of rotation in the dry technique and also by treatments on high-risk patients.

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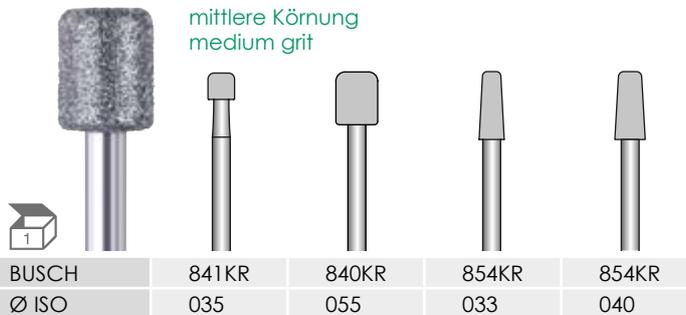
The upper edge of the working part is rounded on all instruments of version KR. This ensures gentle treatment of high-risk patients, whether when smoothing nails or removing callus.

In addition to the time-proven cylindrical basic shapes, we are offering the smaller cylindrical shape 841KR for more delicate treatments. We offer the conical version 854KR in two sizes, so that the right instrument is always available depending on the application and nail size.

Generally we recommend a decrease of rotation in the dry technique and also by treatments on high-risk patients.

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BUSCH & CO. GmbH