PUSH[®]

There is no substitute for quality

PodoinFO

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What is a medical device?

Medical Devices Act

Basis for medical devices (CE marking)

Based on EC directive 93/42/EEC – Medical Devices Directive, or MDD – the Medizinproduktegesetz (Medical Devices Act) was passed into law in Germany in 1994. This law and its associated regulations remained in force until 25 May 2021, when they were replaced by European Regulation (EU)2017/745 of the European Parliament and of the Council, known as the MDR.

BUSCH & CO. GmbH & Co. KG has satisfied the requirements for a compliant quality management system according to the DIN EN ISO 13485 standard since 1998.

The successful implementation of the system is confirmed regularly as part of an annual audit.

By virtue of this procedure, BUSCH & CO. GmbH & Co. KG is legally obliged and entitled to use the CE mark for its medical devices.

All instruments manufactured by BUSCH for medical chiropody / podiatry are medical devices which are identifiable as such from the CE marking and the four-digit number 0124 on the label of each individual package that provide proof of conformity.

In podiatry, brushes and polishers are considered to be cosmetic products, so they do not bear the CE mark.

How should medical devices be treated in podiatric practice?

When authorised for use, medical devices are associated with intended purposes.

For example, the intended purpose of podiatric cutters is to remove excess callus or abrade nail material affected by fungal infection.

Since these actions are therapeutic procedures involving pathologically altered tissue and it is therefore not possible to avoid injury and contact with blood, any such medical device must be sterilisable.

This is the only way to guarantee that hygienic reprocessing (sterilisation) of the medical devices used will effectively prevent cross-contamination between one patient and the next, and prevent pathologically altered tissue from becoming further infected / contaminated by unhygienic instruments. Depending on the invasivity of the therapy performed and the design of the medical device itself, used medical devices are classified as „semi-critical A/B“ or „critical A/B“ and must be sterilised using correspondingly defined reprocessing procedures.

Who must use medical devices?

In general, users can be divided into two groups:

1. In this context, chiropodists / cosmeticians are, by definition, only permitted to work on, treat and dress healthy tissue.
This includes cutting healthy nails, cleaning and outlining healthy nails, removing excess callus, skin care and cosmetic artificial nails.
The use of medical devices is not mandatory for this occupational group.
However, we recommend using the higher grade medical devices if only to avoid risk to yourself as the treatment professional and to your patients.
2. Podiatrists / medical chiropodists carry out healthcare activities (including on pathologically altered tissue) and only earn this entitlement after receiving the corresponding training at a state-approved training facility (school of podiatry).
Such persons are subject to the provisions of the Medizinprodukte-Betreiberverordnung (Ordinance on the Installation, Operation and Use of Medical Devices).
In order to ensure compliance with the Medizinprodukte-Betreiberverordnung and for the safety of patients and users, such activities are conditional upon the use of medical devices.

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All instruments are packaged appropriately to ensure cleanliness and protect them from damage.

Quelles Informations figurent sur l'étiquette du produit?
What kind of information can you find on the product label?

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- We provide each of our packagings with a detailed product label:
- a + b UDI/HIBC-Code c BUSCH Part-No. d1 UDI-Symbol
 - d2 CE-mark
 - d3 symbol for medical device e-g manufacturer's address
 - h LOT-number
 - i max. admissible rpm
 - j material-and shank description
 - k reference symbol, BUSCH-number and size
 - L date of manufacture, advice of further information in BUSCH-catalogues; homepage

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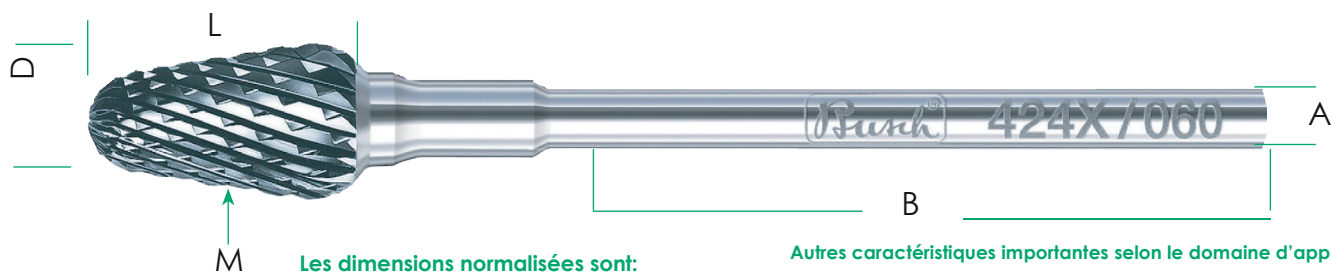
Will all instruments fit my handpiece?

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Yes, because all of our instruments are manufactured according to the **DIN EN ISO 1797-1** standard.

This standard is also applicable to all manufacturers of podiatric drives. Consequently, our instruments are guaranteed to fit in your handpiece perfectly, for both dry and wet techniques.

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The standardised dimensions are:

A - shank diameter Ø 2.334 to 2.350 mm
B - minimum clamping length 30 mm

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Other important features according to application field:

D - working part diameter e.g., 060 = 6.0 mm
L - working part length x mm
M - working part material e.g., carbide

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In general, instruments are distinguished according to the type and material of the working part (grit or blade).

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Instruments with defined grains

- hybrid instruments
- diamond instruments
- ceramic abrasives
- polishers

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Instruments with defined cutting edge

- carbide cutter
- ceramic cutter
- steel cutter

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The combination of abrasive grits - diamond and ceramic - with the proven TWISTER design make the Hybrid TWISTER a high-tech product for modern podiatry.

Despite the essentially finer hybrid grit, it delivers significantly greater abrasion performance in both dry and wet techniques.


At the same time, it leaves behind a smoother surface.

Treatment with the Hybrid TWISTERS is particularly gentle, even when treating diabetic feet and patients with parchment skin. The instruments also have impressively long service lives.

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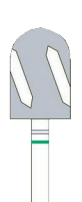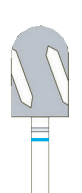
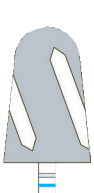
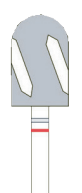
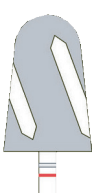
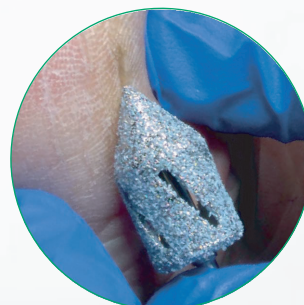
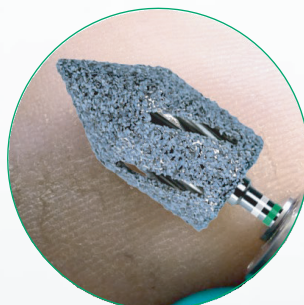
Our expert for special cases

Remove large callus as well as rhagades, localised callus (e.g. on the tips of the toes) and corns with a single instrument.

The special feature of this Twister shape – the long, parallel head with a tapered tip – allows you to work both on larger areas and between the toes.

This makes the treatment process easier, and reduces both the need to change instruments and the time spent on device reprocessing.

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one-piece production ensures optimum concentricity properties

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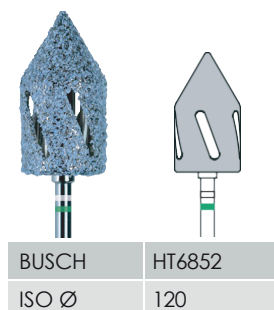
... the modern grit - diamond plus ceramic - is coated on a durable stainless steel cap.

Efficiency:
one HybridCap instead of hundreds

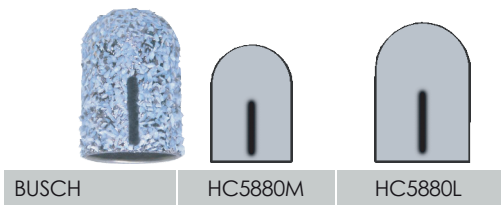
Sustainability:
instead of throwaway

Patient friendliness:
smoother surfaces in less time


Hygiene:
HybridCap and Cap Support are disinfected and sterilizable.
A CE-symbol medical product made by BUSCH

Cap and mandrel have to be disinfected and sterilized unmounted for hygienic and technical reasons.

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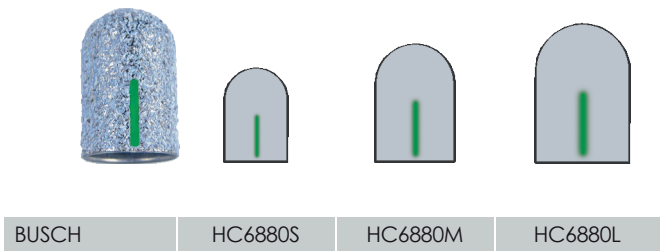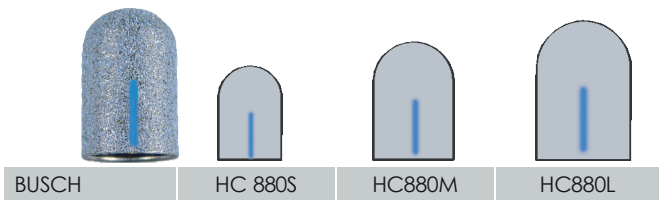
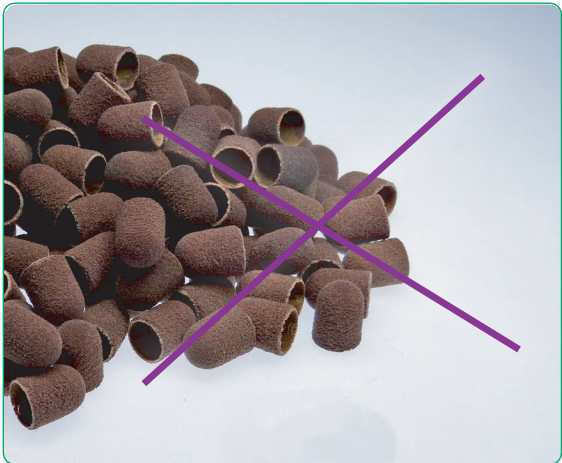
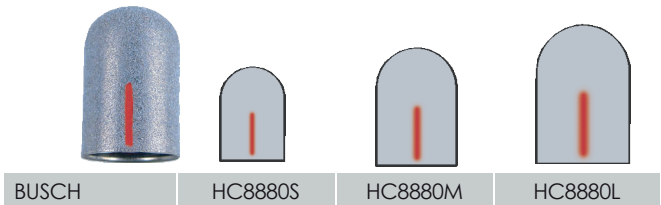
... only one cap
instead of hundreds...

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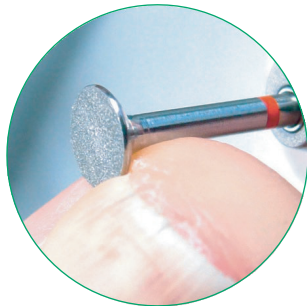
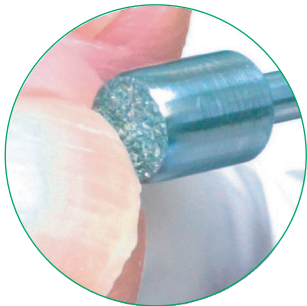
Cap support long-lasting and exact fitting

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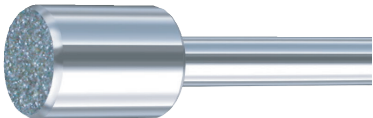
Extremely gentle nail shortening

Thanks to their intelligent design, the TOP Grip series are indispensable instruments for the targeted grinding of damaged, unnaturally thickened and deformed nails.

TOP Grip instruments enable nails to be shortened safely at the front and sides or the nail plate to be reduced down to the nail fold. The instruments have a smooth bevel with no diamond coating in the transition region to the diamond-coated front face.

With careful use, treatments of any kind can be completed without injury, which is essential, particularly when dealing with diabetics.

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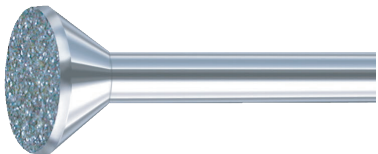
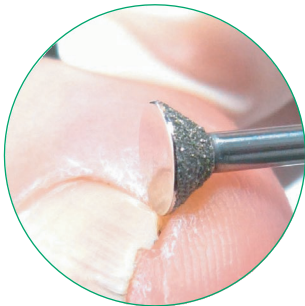
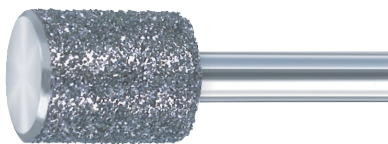
Treating nail wall regions

The SIDE Grip series are indispensable instruments for the targeted grinding of damaged or thickened nails, or critical areas of skin.

Only the lateral area of the working part is coated with diamonds, and the bevel in the transition region to the flat front face is smooth, with no diamond coating, so SIDE Grip instruments can be used to treat wall areas safely, with no unintended grinding contact.

With careful use, treatments of any kind can be completed without injury, which is essential, particularly when dealing with diabetics.

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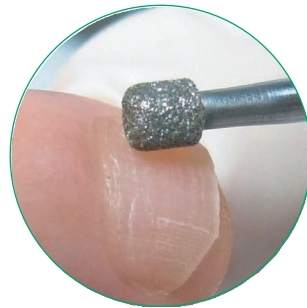
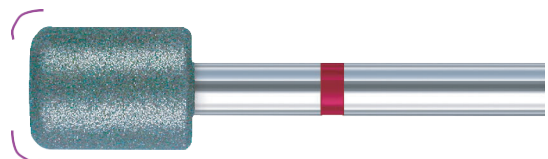
Gentle treatment of damaged nails

Rounded edges are the instantly recognisable feature of the KR series.


This enables extremely gentle smoothing of nails or abrasion of callus, particularly when dealing with vulnerable patients.

With careful use, treatments of any kind can be completed without injury, which is essential, particularly when dealing with diabetics.

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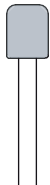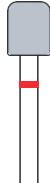
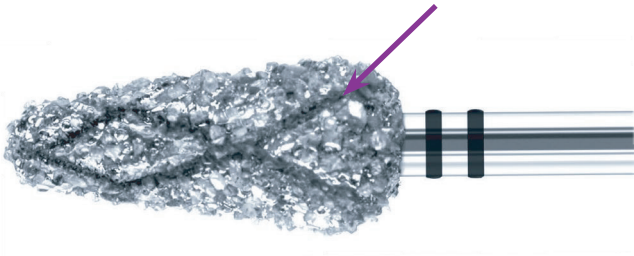
Abrasion of thick calluses and removal of hard calluses


The CD instruments with peripheral cooling groove technology for cool and pleasantly gentle work using dry and wet techniques.


			
		Grain méga grossier mega coarse grit	
			
BUSCH	CD4894	CD4369	
ISO Ø	070	090	

		● Grain super grossier super coarse grit		
				
BUSCH	CD5893	CD5893	CD5894	CD5892
ISO Ø	050	065	065	075

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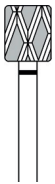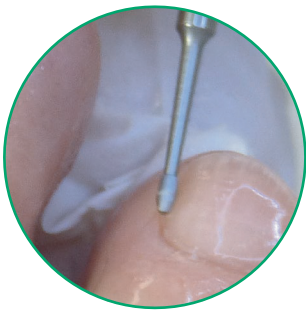Gentle cleaning of the nail fold and removal of the cuticle in cases of heavier callus

In addition to the familiar classic ONY CLEAN the diamond-coated variant is particularly suitable for heavier cornification of the nail fold.

This variant is also particularly effective in areas calling for substantial abrasion in a closely defined location.

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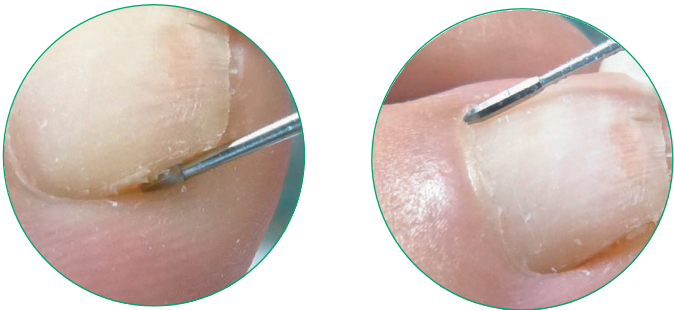
Gentle cleaning of the nail fold and removal of the cuticle

The special instrument made of stainless steel with the proven polygonal working part, without blades, which cleans the nail fold gently and leaves remarkably smooth nail surfaces.

The elongated shape of the working part is ideal for wide-area work.

Caution: Coarse diamond instruments or instruments with cuts used for this application will create unwanted striations on the nail.

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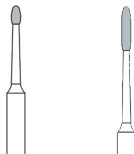
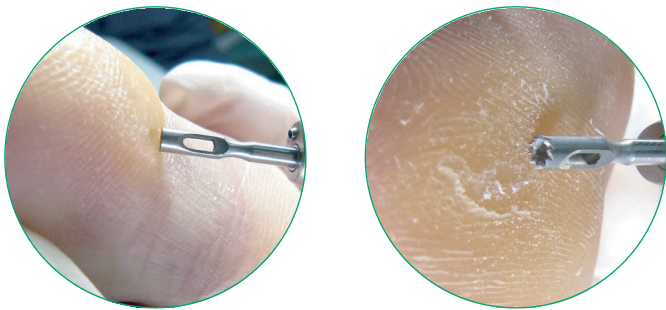
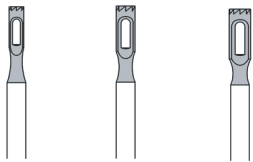
Gentle removal
of corns/clavi
and callus, grinding
of callus grooves

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

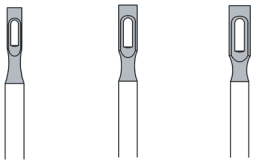
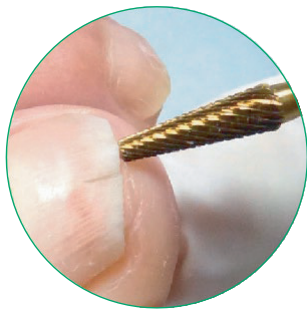
With their golden titanium nitride coating, they ensure fast, safe working in gel and acrylic in the nail prosthesis and on thickened nails

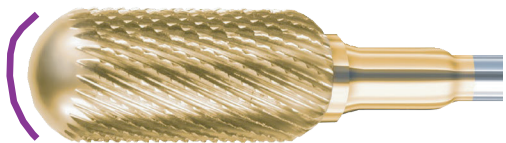The particular advantages of this carbide cutter series are the highly effective SPEED cuts, which ensure rapid substance abrasion, and the smooth, domed working part tip without cuts to prevent injuries.

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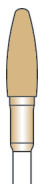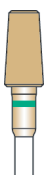
SPEED PLUS cut
(green ring)
• front without cuts

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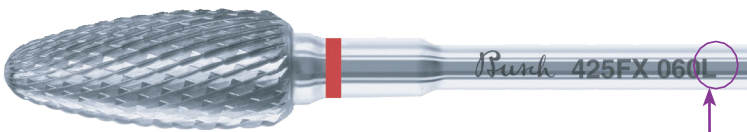
Cutters for left-handed operators

We have introduced left-handed versions of the popular cutters into our range in order to offer left-handed users the same easy-to-use and convenient carbide cutters as right-handed users.


These carbide cutters can be identified from the labelling on the shank. They are marked with L at the end of the figure number.

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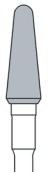
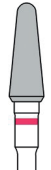
denture croisée fine
fine double cut

BUSCH	425FX060L	429FX045L	429FX060L
ISO Ø	060	045	060

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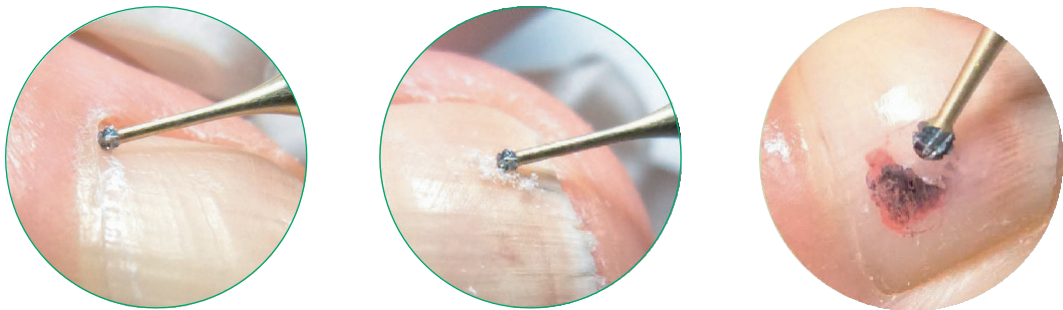
Soft removal of clavi and
nail fungus as well as
treatment of gryphotic nails
and periungual callus

The slimline neck construction of this
carbide cutter enables a better overview
of the treatment site.

The special cut arrangement is instantly
recognisable by the green shank ring
and gold plated neck.

							
BUSCH	1SXM	1SXM	1SXM	1SXM	1SXM	1SXM	1SXM
ISO Ø	010	012	014	016	018	021	023

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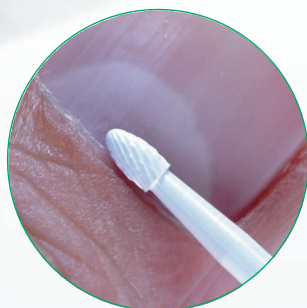All in white !

The filigree shapes entirely made of ceramic; for gentle treatment and precise shaping with an optimum overview.

This aesthetic material stands out through its excellent fracture toughness and hardness which, in combination with the proven BUSCH cuts, yields the typical attractive results.

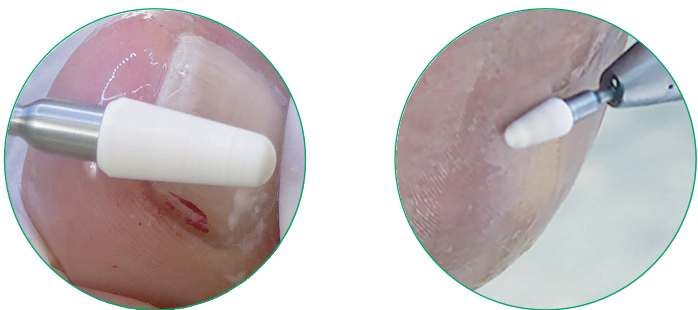
The high-performance ceramic material used for KERA cutters is up to 25% respectively up to 60% lighter compared with carbide, thus permitting more gentle cutting – a tangible advantage for both patient and user when treating diseased natural nails as well as artificial nails.

Patients who are highly allergic to metals welcome the additional bio-compatibility. Naturally, KERA cutters are medical devices and suitable for all approved disinfection and sterilization methods.

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SPEED cut, with passive safety,
recommended for diabetic patients

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From pre-polishing to final polish, we offer shapes for practically all applications:

- polishing of the nail surface in nail prosthetics
- polishing of acrylates and nail prosthesis
- rounding and fine finishing of the nail edges
- smoothening sensitive skin areas
- smoothening of rough surfaces of natural nails and/or gel/acrylics

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smoothing, green

SiC, Silicone, Grit size coarse
bond hardness medium


BUSCH	9626G	9626G	9626G	9628G	9628G	9627G	9629G
ISO Ø	055	100	120	065	100	080	080

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polishing, anthrazite

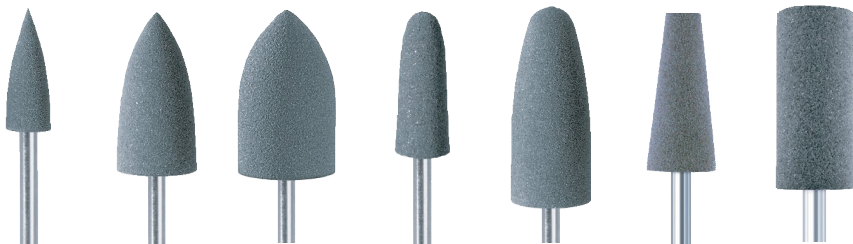SiC, Silicone, Grit size medium
bond hardness medium

BUSCH	9626	9626	9626	9628	9628	9627	9629
ISO Ø	055	100	120	065	100	080	080

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end-polishing, khaki

SiC, Silicone, Grit size fine
bond hardness medium

BUSCH	9622	9622	9622	9624	9624	9623	9625
ISO Ø	055	100	120	065	100	080	080

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high-grade corundum,
ceramic bonded

- for the removal of particles
sticking to diamond instruments
(grit size fine and coarser)
at an instrument
speed of 1.000-5.000 rpm.
- dampen before use

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coarse grit

- for dressing of polishers
and abrasives

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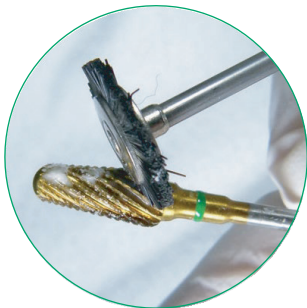
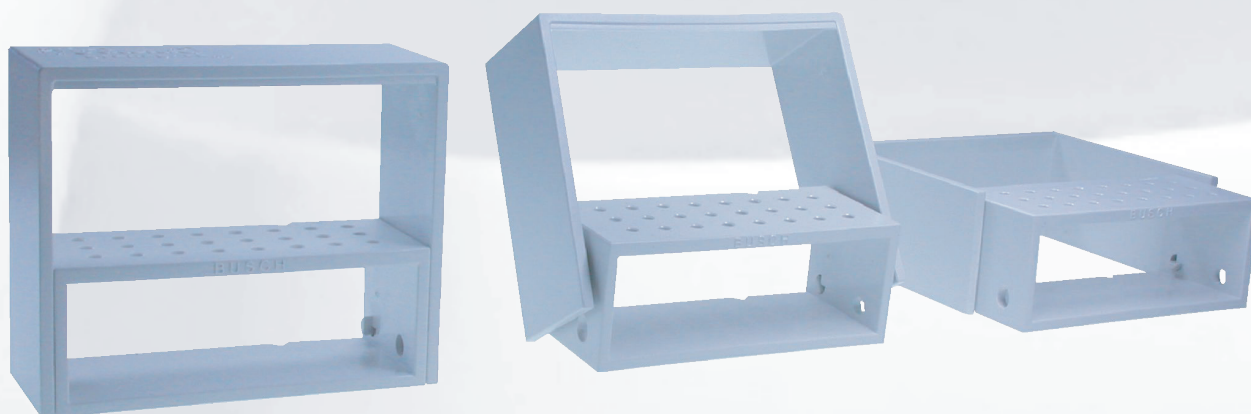
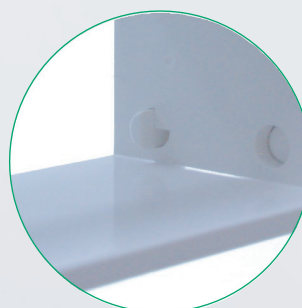
clever and hygienic


- robust overall design with improved stability
- rounded-off safety edges
- optimized opening and closing mechanism
- additional mounting options
- concentricity tester „control“
- for sterilization up to 180 °C
- for an individual and clear arrangement of instrument sets
- for the safekeeping of instruments before, during and after disinfection, cleaning and or sterilization
- for all usual methods of disinfection, cleaning and sterilization

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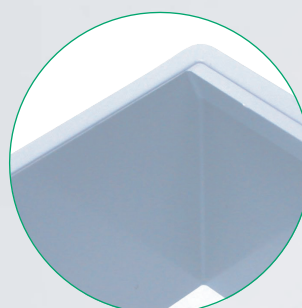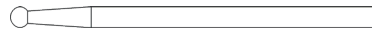
dimensions: 75 x 29 x 62 mm

material: medical resin

for 24 handpiece instruments shank HP Ø 2,35 mm

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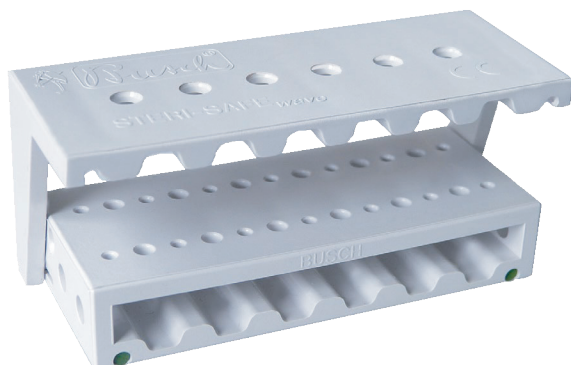
hygiene box PODO-LOCK

for the inclusion of a STERI-SAFEpro

Busch-No.: HPL 805

- securely lockable
- fillable with disinfectant for pre-cleaning
- dust-tight storage of disinfected/sterilized instrument sets
- not sterilizable

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
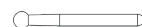
dimensions: 76 x 30 x 33 mm

material: medical resin

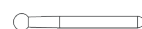for 14 instruments for the turbine shank FG Ø 1,60 mm

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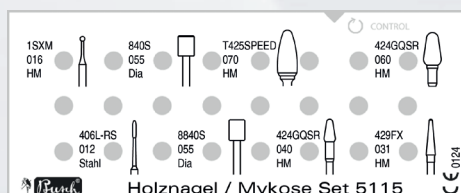
Basic set for precise, quick yet gentle treatment of mycotic / gryptic nails and for removing large quantities of nail substance

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All Busch sets are printed with the outline drawings, Busch no. and ISO size of the instruments that belong to the set.

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Thoroughly systematic working

procedures, hygiene and organization are important for treating patients safely with rotary instruments.

systematic

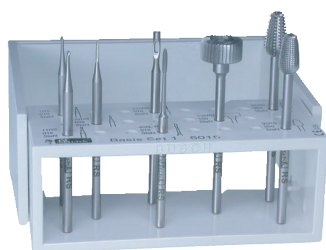
To support your working procedures, we have designed instrument sets for specific applications. The configuration of these instrument sets is considered a recommendation only, and can be complemented as required. The sets are delivered in the hygiene box PODO-LOCK.

hygiene

The Steri-SafePro is a universal bur holder for storing and organizing burs as well as being perfect for disinfecting, cleaning and sterilizing specific instruments. The instruments remain in the STERI-SAFEPro throughout the hygiene procedure.

organization


Practical instrument sets, enable the provision of hygienically prepared instruments, which can be transferred to the hygienic cycle after use, in preparation for the next treatment.

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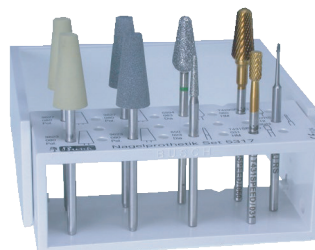
Stainless steel instruments for treating the nail fold, corns/clavi and for removing callus.

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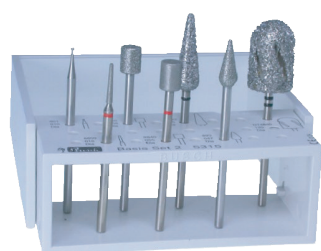
For specific treatment of diseased natural nails and artificial nail material in nail prosthetics.

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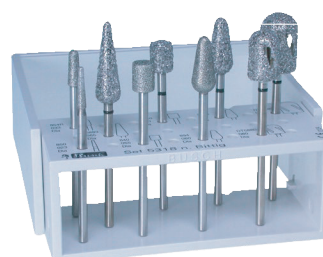
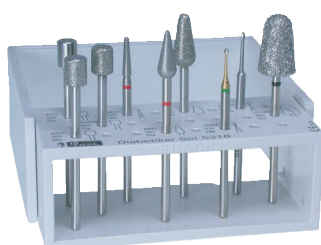
According to the instruction and grinding principles published by Fritz Bittig. Usable in wet and dry techniques.

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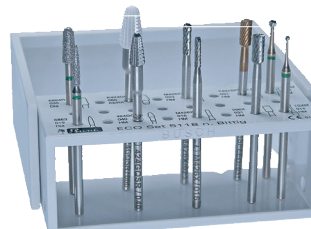
For minimum invasive treatment of the diabetic foot; rounded instruments with only partial diamond coating minimize the risk of injury during treatment.

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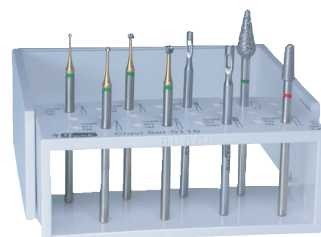For grinding technique on skin and nails as indicated by Bittig.

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For specifically treating and removing corns.

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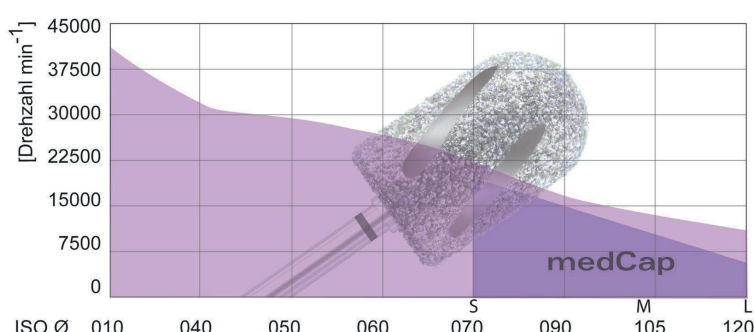
The abovementioned sets are delivered in a hygiene box. The box can be closed firmly and provides protection from dust.

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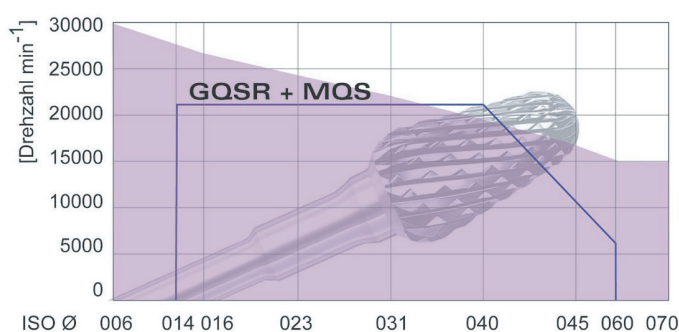
Instruments en carbure et fraises KERA Carbide cutters and KERA cutters

max. admissible speed

The mentioned maximum admissible speed (rpm) indicated on each BUSCH pack is not allowed to be exceeded due to safety regulations.

speed ranges

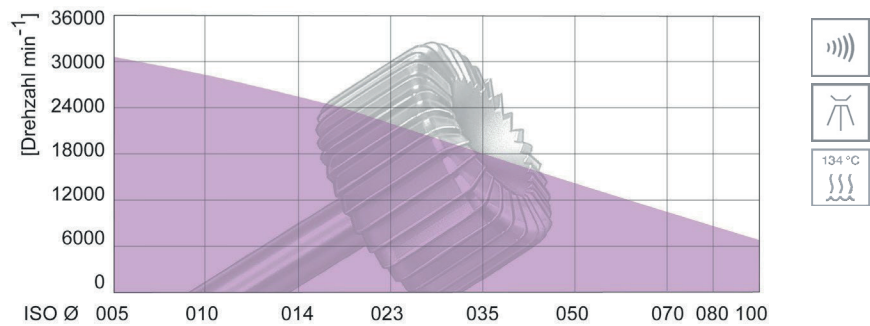
Keeping within the instrument-specific speed ranges leads to good treatment results (see diagram).

If the diameter of the working part exceeds that of the shank, powerful centrifugal forces may build up at higher speed which might bend the shank and/or fracture the instrument.


Rule of thumb: The larger the diameter of the instrument, the slower its speed.

Lower speeds should be selected in dry technique to keep any heat that might be generated to a minimum.

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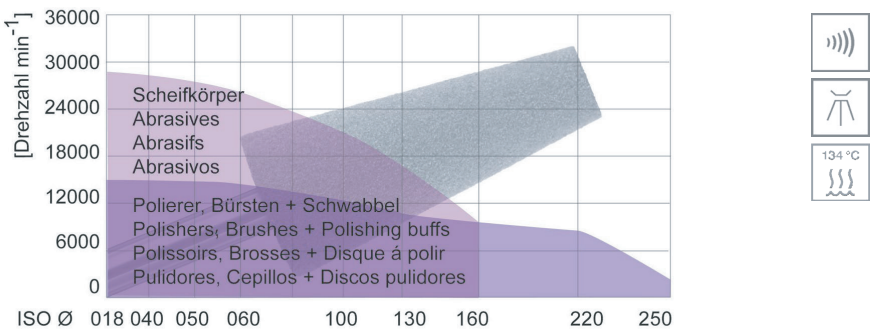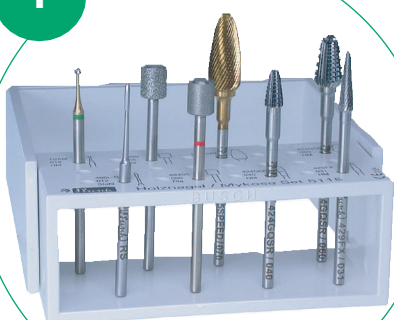
The use of the STERI-SAFEpro during hygienic reprocessing prevents your instruments from being damaged by knocking against each other.

Hygienically reprocessed instruments on the disinfectable and sterilisable STERI-SAFEpro instrument tray.

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Carbide cutter with speed cuts during treatment. (Example)

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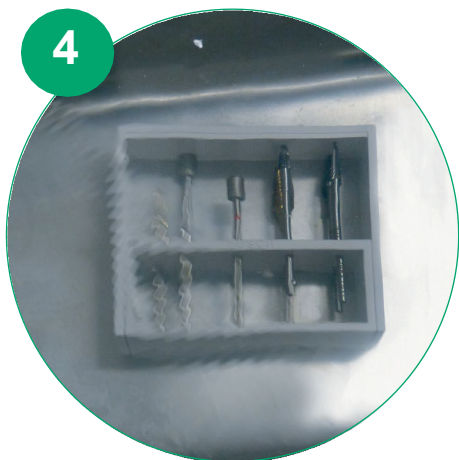
Instruments in the STERI-SAFEpro

As soon as the treatment has ended, remove any heavy soiling, place the instruments in the STERI-SAFEpro for preliminary disinfection, and store in the HPL box filled with disinfection fluid.

The mixing proportion of disinfection concentrates as well as the max. acting time as specified in the manufacturer's specifications must be observed meticulously.

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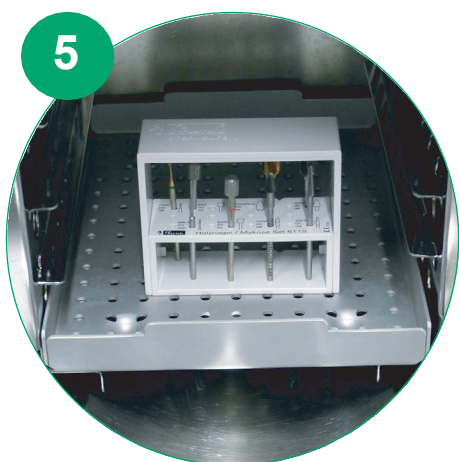
Instruments in the STERI-SAFEpro during cleaning in the ultrasonic bath with cleaning additive. Then rinse with water and dry. If instruments are heavily soiled, it is advisable to carry out pre-cleaning by hand. For diamond instruments (grain size fine and coarser), use of the cleaning stone 599 is recommended.

Alternatively: Use of a cleaning/disinfection (RDG) unit, optionally with thermal disinfection.

Cleanliness check (with 6-8x magnifier), clean again if necessary!

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Instruments in the STERI-SAFEpro in the autoclave.

No sterilisation foil = Thermal disinfection

With sterilisation foil = Sterilisation

All instruments are designed to be sterilised with steam.

Exception: Polishers and brushes must not be subjected to steam sterilisation.

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After complete thermal disinfection, instruments are stored and protected from dust in the STERI-SAFEpro in storage box HPL 805.

After complete thermal sterilisation, instruments are stored accordingly in the STERI-SAFEpro in sterilisation foil.

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The current reprocessing recommendations for instruments can always be found at www.busch.eu.

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- select appropriate qualified cleaners/ disinfectants giving special regard to material friendliness; do not use any household detergents; do not use acids or alkaline solutions
- use consistent dosing (no overdosing) according to the manufacturer's specifications; take evaporation into account when using cleaner/ disinfectant baths for several days (not only water evaporates, the concentration of chemical ingredients will increase), ready-to-use solutions are preferable compared to concentrates
- comply with the maximum permissible dwell times in cleaners/disinfectants according to the manufacturer's specifications

Failure to observe this warning may result in corrosion of the instruments, which will affect their performance, stability and surface texture requiring them to be discarded prematurely.

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