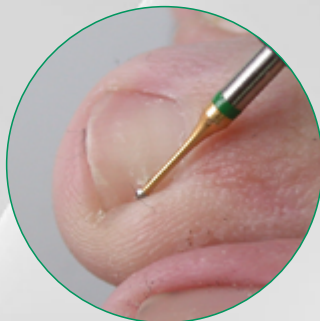
Rush[®]

There is no substitute for quality

Podoinfo

Rotierende Instrumente in Praxis und Ausbildung
Rotary instruments for practice and education

Christian Rothe

Gert Busch

There is no substitute for quality

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„There is no substitute for quality“ this conviction has committed our company's philosophy since our foundation in 1905.

Our ideas and all processes of research and development, production and distribution meet with the demands of this philosophy.

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What is a medical device?

Medical Devices Act

Basis for medical devices (CE marking)

Based on EC directive 93/42/EEC – Medical Devices Directive, or MDD – the Medizinproduktegesetz (Medical Devices Act) was passed into law in Germany in 1994. This law and its associated regulations remained in force until 25 May 2021, when they were replaced by European Regulation (EU)2017/745 of the European Parliament and of the Council, known as the MDR.
BUSCH & CO. GmbH & Co. KG has satisfied the requirements for a compliant quality management system according to the DIN EN ISO 13485 standard since 1998.
The successful implementation of the system is confirmed regularly as part of an annual audit.
By virtue of this procedure, BUSCH & CO. GmbH & Co. KG is legally obliged and entitled to use the CE mark for its medical devices.
All instruments manufactured by BUSCH for medical chiropody / podiatry are medical devices which are identifiable as such from the CE marking and the four-digit number 0124 on the label of each individual package that provide proof of conformity.
In podiatry, brushes and polishers are considered to be cosmetic products, so they do not bear the CE mark.

Who must use medical devices?

In general, users can be divided into two groups:

1. In this context, chiropodists / cosmeticians are, by definition, only permitted to work on, treat and dress healthy tissue.
This includes cutting healthy nails, cleaning and outlining healthy nails, removing excess callus, skin care and cosmetic artificial nails.
The use of medical devices is not mandatory for this occupational group.
However, we recommend using the higher grade medical devices if only to avoid risk to yourself as the treatment professional and to your patients.
2. Podiatrists / medical chiropodists carry out healthcare activities (including on pathologically altered tissue) and only earn this entitlement after receiving the corresponding training at a state-approved training facility (school of podiatry).
Such persons are subject to the provisions of the Medizinprodukte-Betreiberverordnung (Ordinance on the Installation, Operation and Use of Medical Devices).
In order to ensure compliance with the Medizinprodukte-Betreiberverordnung and for the safety of patients and users, such activities are conditional upon the use of medical devices.

How should medical devices be treated in podiatric practice?

When authorised for use, medical devices are associated with intended purposes.
For example, the intended purpose of podiatric cutters is to remove excess callus or abrade nail material affected by fungal infection.
Since these actions are therapeutic procedures involving pathologically altered tissue and it is therefore not possible to avoid injury and contact with blood, any such medical device must be sterilisable.
This is the only way to guarantee that hygienic reprocessing (sterilisation) of the medical devices used will effectively prevent cross-contamination between one patient and the next, and prevent pathologically altered tissue from becoming further infected / contaminated by unhygienic instruments.
Depending on the invasivity of the therapy performed and the design of the medical device itself, used medical devices are classified as „semi-critical A/B“ or „critical A/B“ and must be sterilised using correspondingly defined reprocessing procedures.

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All instruments are packaged
appropriately to ensure cleanliness
and protect them from damage.

Welche Informationen beinhaltet das Produktetikett?
What kind of information can you find on the product label?

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We provide each of our
packagings with a detailed
product label:

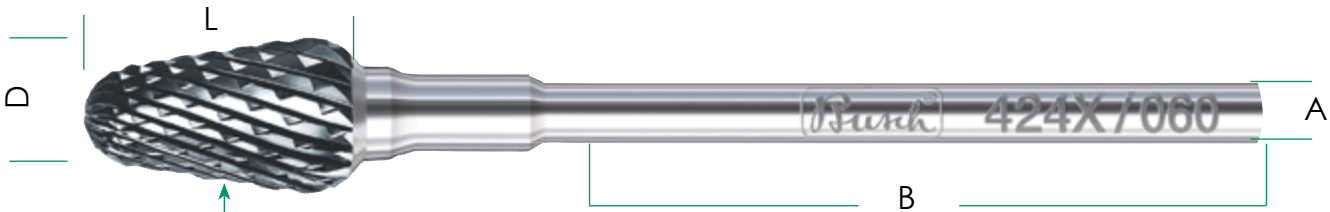
a + b UDI/HIBC-Code
c BUSCH Part-No.
d1 UDI-Symbol
d2 CE-mark
d3 symbol for medical device
e-g manufacturer's address
h LOT-number
i max. admissible rpm
j material-and shank
description
k reference symbol,
BUSCH-number and size
L date of manufacture,
advice of further information
in BUSCH-catalogues;
homepage

Passen alle Instrumente
in meinen Antrieb?
Will all instruments fit
my handpiece?

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Yes, because all of our instruments
are manufactured according to
the **DIN EN ISO 1797-1**
standard.
This standard is also applicable
to all manufacturers of podi-
atric drives. Consequently, our
instruments are guaranteed to fit
in your handpiece perfectly, for
both dry and wet techniques.

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The standardised dimensions are:
A - shank diameter Ø 2.334 to 2.350 mm
B - minimum clamping length 30 mm

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Other important features according to application field:
D - working part diameter e.g., 060 = 6.0 mm
L - working part length x mm
M - working part material e.g., carbide

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In general, instruments are
distinguished according to the
type and material of the working
part (grit or blade).

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**Instruments with
defined grains**

- hybrid instruments
- diamond instruments
- ceramic abrasives
- polishers

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**Instruments with
defined cutting edge**

- carbide cutter
- ceramic cutter
- steel cutter

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Abrasives

- for smoothing nail surfaces, calluses and artificial nails
- Surface renewal during use. The initial shape is diminished by wear, but „fresh“ grits are always available.

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
Polishers

- similar to the abrasive tools, but the abrasive grit is much more elastic and more flexible bonded
- polishing of nail surfaces or artificial nails as part of the nail prosthesis.

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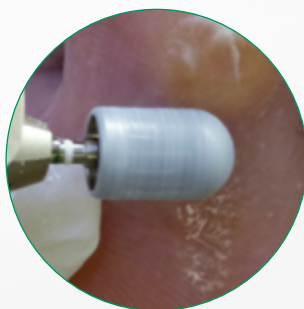The combination of abrasive grits - diamond and ceramic - with the proven TWISTER design make the Hybrid TWISTER a high-tech product for modern podiatry.

Despite the essentially finer hybrid grit, it delivers significantly greater abrasion performance in both dry and wet techniques. At the same time, it leaves behind a smoother surface.

Treatment with the Hybrid TWISTERS is particularly gentle, even when treating diabetic feet and patients with parchment skin. The instruments also have impressively long service lives.

- ● **super grobe Hybridkörnung** Abräumen schwerer Verhornungen
super coarse grit removal of severe callus

					
BUSCH		HT5880	HT5880		HT5854R
ISO Ø		105	125		105


- ● **grobe Hybridkörnung** Abtrag dicker Hornschwielen
coarse grit removal of thick callus


					
BUSCH	HT6880	HT6880	HT6880		HT6854R
ISO Ø	080	100	120		100

- ● **mittlere Hybridkörnung** Entfernung starker Verhornungen
medium grit removal of hard callus

					
BUSCH	HT880	HT880	HT880		HT854R
ISO Ø	075	095	115		095

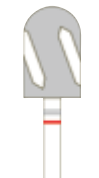- ● **feine Hybridkörnung** Entfernung beginnender Verhornungen
fine grit removal of incipient callus

					
BUSCH	HT8880	HT8880	HT8880		HT8854R
ISO Ø	075	095	115		095

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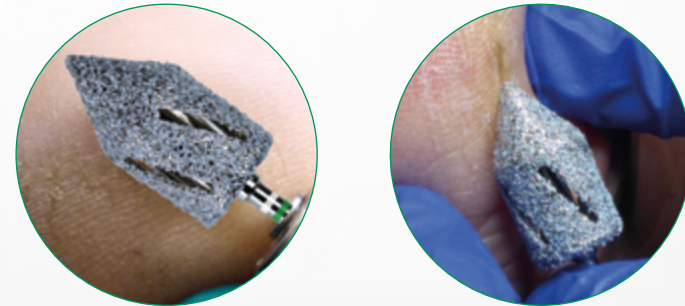Our expert for special cases

Remove large callus as well as rhagades, localised callus (e.g. on the tips of the toes) and corns with a single instrument.

The special feature of this Twister shape - the long, parallel head with a tapered tip - allows you to work both on larger areas and between the toes.

This makes the treatment process easier, and reduces both the need to change instruments and the time spent on device reprocessing.

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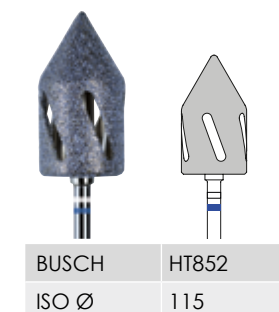
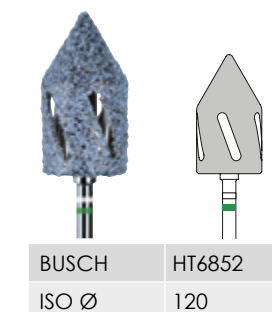
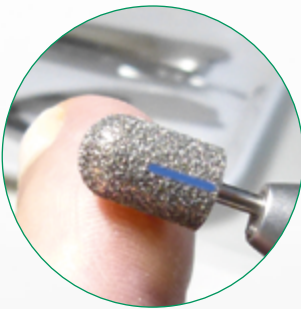
coarse grit
removal of thick callus
for treatment of severe rhagades

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medium grit
removal of hard callus
for treatment of rhagades

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... the modern grit - diamond plus ceramic - is coated on a durable stainless steel cap.

Efficiency:

one HybridCap instead of hundreds

Sustainability:

instead of throwaway

Patient friendliness:

smoother surfaces in less time

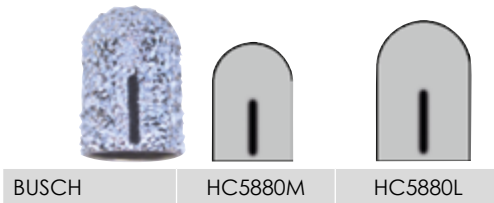
Hygiene:

HybridCap and Cap Support are disinfected and sterilizable.


A CE-symbol medical product made by BUSCH

Cap and mandrel have to be disinfected and sterilized unmounted for hygienic and technical reasons.


- **super grobe Körnung**, Abräumen schwerer Verhornungen
super coarse grit, removal of severe callus

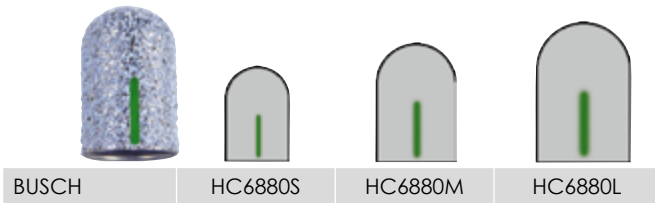
- **grobe Körnung**, Abtragen dicker Hornschwielen
coarse grit, removal of thick callus

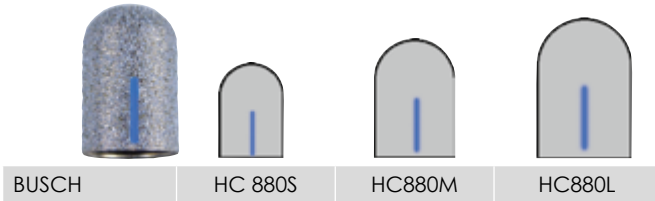- **mittlere Körnung**, Entfernen starker Verhornungen
medium grit, removal of hard callus

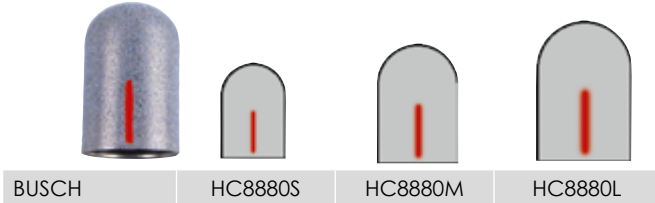- **feine Körnung**, Entfernen beginnender Verhornungen
fine grit, removal of incipient callus

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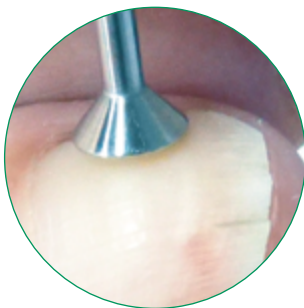
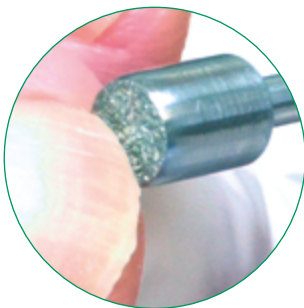
medCap and Cap support **must** separately disinfected and sterilized.

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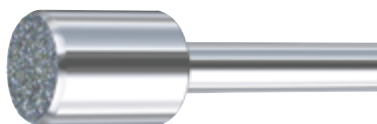
Extremely gentle nail shortening

Thanks to their intelligent design, the TOP Grip series are indispensable instruments for the targeted grinding of damaged, unnaturally thickened and deformed nails.

TOP Grip instruments enable nails to be shortened safely at the front and sides or the nail plate to be reduced down to the nail fold.

The instruments have a smooth bevel with no diamond coating in the transition region to the diamond-coated front face.

With careful use, treatments of any kind can be completed without injury, which is essential, particularly when dealing with diabetics.

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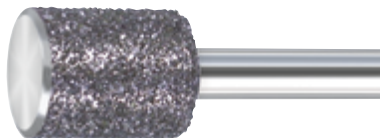
Treating nail wall regions

The SIDE Grip series are indispensable instruments for the targeted grinding of damaged or thickened nails, or critical areas of skin.

Only the lateral area of the working part is coated with diamonds, and the bevel in the transition region to the flat front face is smooth, with no diamond coating, so SIDE Grip instruments can be used to treat wall areas safely, with no unintended grinding contact.

With careful use, treatments of any kind can be completed without injury, which is essential, particularly when dealing with diabetics.

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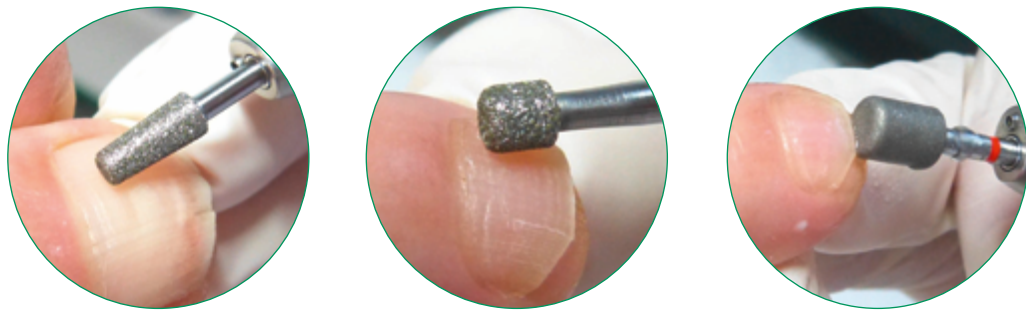
Gentle treatment of damaged nails

Rounded edges are the instantly recognisable feature of the KR series.

This enables extremely gentle smoothing of nails or abrasion of callus, particularly when dealing with vulnerable patients.

With careful use, treatments of any kind can be completed without injury, which is essential, particularly when dealing with diabetics.

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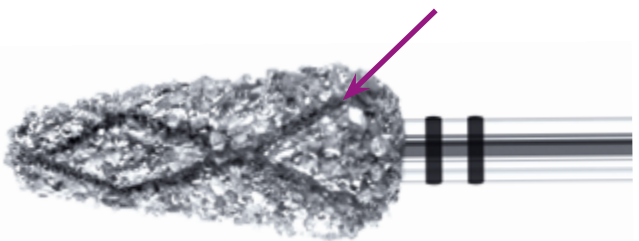
Abrasion of thick calluses and removal of hard calluses

The CD instruments with peripheral cooling groove technology for cool and pleasantly gentle work using dry and wet techniques.

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Gentle cleaning of the nail fold and removal of the cuticle in cases of heavier callus

In addition to the familiar classic ONLY CLEAN the diamond-coated variant is particularly suitable for heavier cornification of the nail fold.

This variant is also particularly effective in areas calling for substantial abrasion in a closely defined location.

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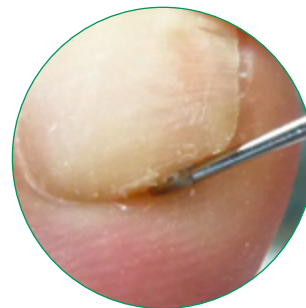
Gentle cleaning of the nail fold and removal of the cuticle

The special instrument made of stainless steel with the proven polygonal working part, without blades, which cleans the nail fold gently and leaves remarkably smooth nail surfaces.


The elongated shape of the working part is ideal for wide-area work.

Caution: Coarse diamond instruments or instruments with cuts used for this application will create unwanted striations on the nail.

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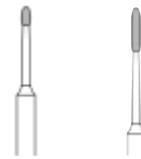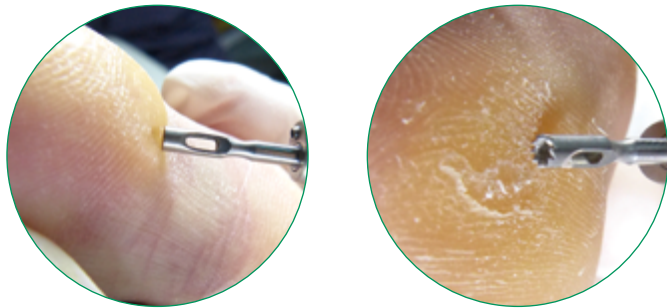
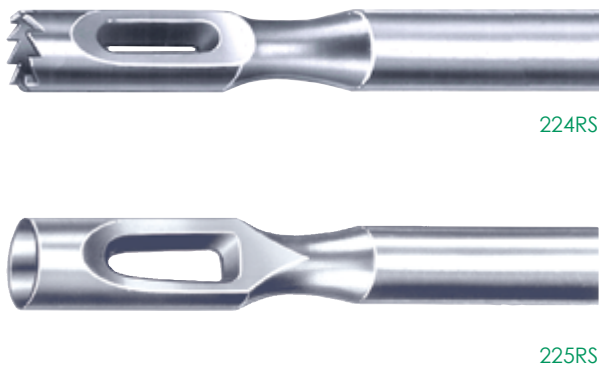
Gentle removal
of corns/clavi
and callus, grinding
of callus grooves


			
BUSCH	224RS	224RS	224RS
ISO Ø	018	023	027

			
BUSCH	225RS	225RS	225RS
ISO Ø	018	023	027

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
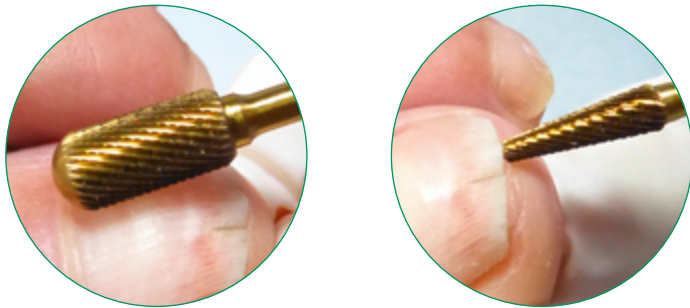
With their golden titanium
nitride coating, they
ensure fast, safe working
in gel and acrylic in
the nail prosthesis and on
thickened nails

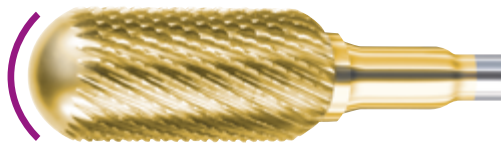
The particular advantages of this
carbide cutter series are the highly
effective SPEED cuts, which ensure
rapid substance abrasion, and the
smooth, domed working part tip
without cuts to prevent injuries.

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

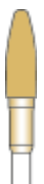
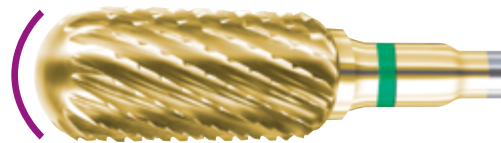
SPEED PLUS cut
(green ring)
• front without cuts

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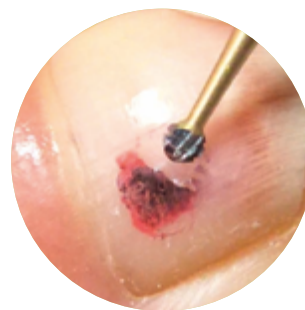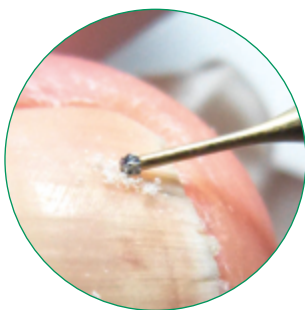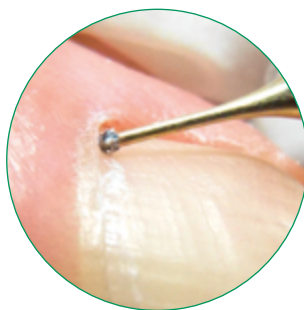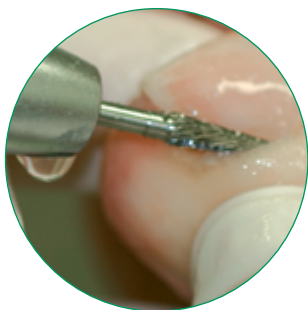Cutters for left-handed operators

We have introduced left-handed versions of the popular cutters into our range in order to offer left-handed users the same easy-to-use and convenient carbide cutters as right-handed users.

These carbide cutters can be identified from the labelling on the shank. They are marked with L at the end of the figure number.

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
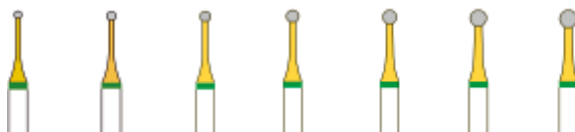
Soft removal of clavi and nail fungus as well as treatment of gryphotic nails and periungual callus

The slimline neck construction of this carbide cutter enables a better overview of the treatment site.

The special cut arrangement is instantly recognisable by the green shank ring and gold plated neck.

BUSCH	1SXM	1SXM	1SXM	1SXM	1SXM	1SXM	1SXM
ISO Ø	010	012	014	016	018	021	023

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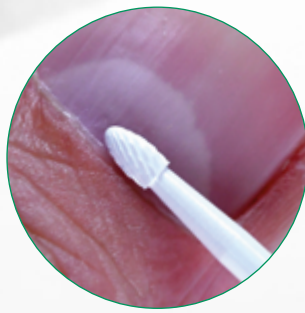All in white!

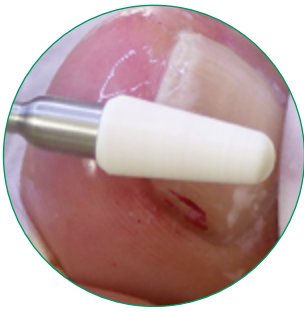
The filigree shapes entirely made of ceramic; for gentle treatment and precise shaping with an optimum overview.

This aesthetic material stands out through its excellent fracture toughness and hardness which, in combination with the proven BUSCH cuts, yields the typical attractive results.

The high-performance ceramic material used for KERA cutters is up to 25% respectively up to 60% lighter-compared with carbide, thus permitting more gentle cutting – a tangible advantage for both patient and user when treating diseased natural nails as well as artificial nails.


Patients who are highly allergic to metals welcome the additional bio-compatibility. Naturally, KERA cutters are medical devices and suitable for all approved disinfection and sterilization methods.

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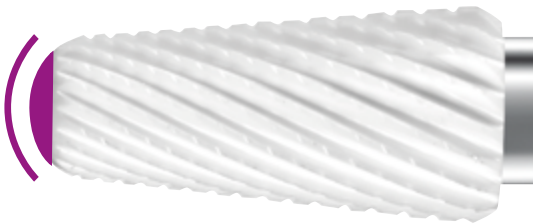Special SPEED cut,
with passive safety,
recommended for diabetic patients

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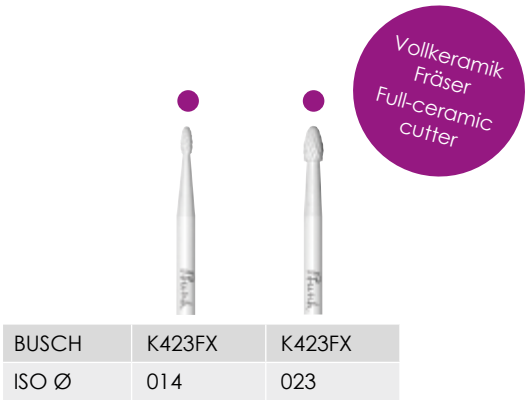
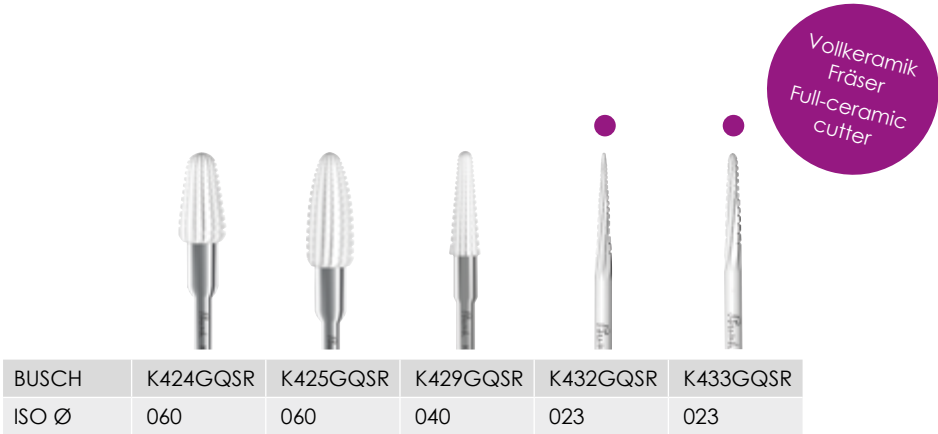
From pre-polishing
to final polish, we offer
shapes for practically
all applications:

- polishing of the nail surface in nail prosthetics
- polishing of acrylates and nail prosthesis
- rounding and fine finishing of the nail edges
- smoothing sensitive skin areas
- smoothing of rough surfaces of natural nails and/or gel/acrylics

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smoothing, green

SiC, Silicone, Grit size coarse
bond hardness medium

BUSCH	9626G	9626G	9626G	9628G	9628G	9627G	9629G
ISO Ø	055	100	120	065	100	080	080

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polishing, anthrazite

SiC, Silicone, Grit size medium
bond hardness medium

BUSCH	9626	9626	9626	9628	9628	9627	9629
ISO Ø	055	100	120	065	100	080	080

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end-polishing, khaki

SiC, Silicone, Grit size fine
bond hardness medium

BUSCH	9622	9622	9622	9624	9624	9623	9625
ISO Ø	055	100	120	065	100	080	080

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high-grade corundum,
ceramic bonded

- for the removal of particles sticking to diamond instruments (grit size fine and coarser) at an instrument speed of 1.000-5.000 rpm.
- dampen before use

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coarse grit

- for dressing of polishers and abrasives

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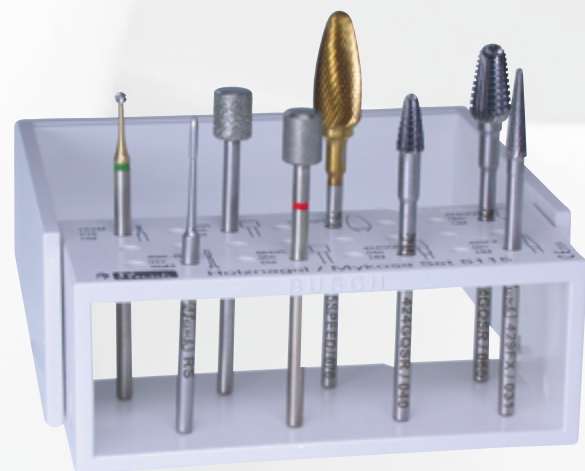
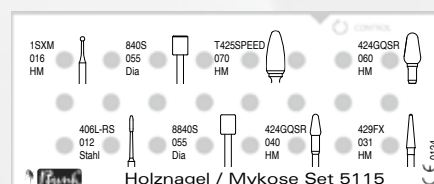
Basic set for precise, quick yet gentle treatment of mycotic / gryptic nails and for removing large quantities of nail substance

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All Busch sets are printed with the outline drawings, Busch no. and ISO size of the instruments that belong to the set.

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Thoroughly systematic working procedures and organization are important for treating patients safely with rotary instruments.

systematic

To support your working procedures, we have designed instrument sets for specific applications. The configuration of these instrument sets is considered a recommendation only, and can be complemented as required. The sets are delivered in the hygiene box PODO-LOCK.

organization

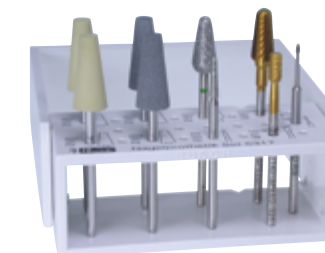
Practical instrument sets, enable the provision of hygienically prepared instruments, which can be transferred to the hygienic cycle after use, in preparation for the next treatment.

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
Stainless steel instruments for treating the nail fold, corns/clavi and for removing callus.

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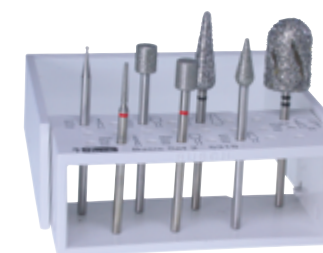For specific treatment of diseased natural nails and artificial nail material in nail prosthetics.

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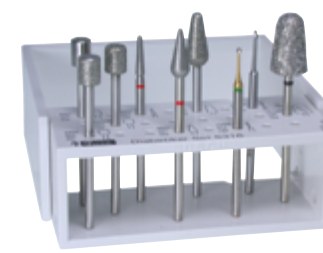
Diamond instruments in various grit sizes for gently grinding callus and nails.

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According to the instruction and grinding principles published by Fritz Bittig. Usable in wet and dry techniques.

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For minimum invasive treatment of the diabetic foot; rounded instruments with only partial diamond coating minimize the risk of injury during treatment.

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For specifically treating and removing corns.

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
The abovementioned sets are delivered in a hygiene box. The box can be closed firmly and provides protection from dust.

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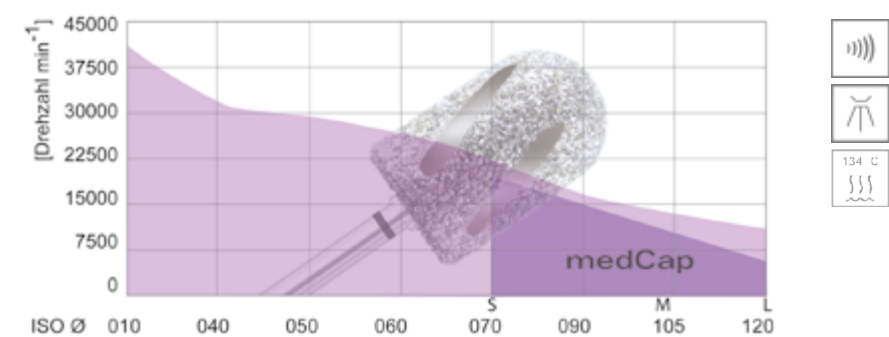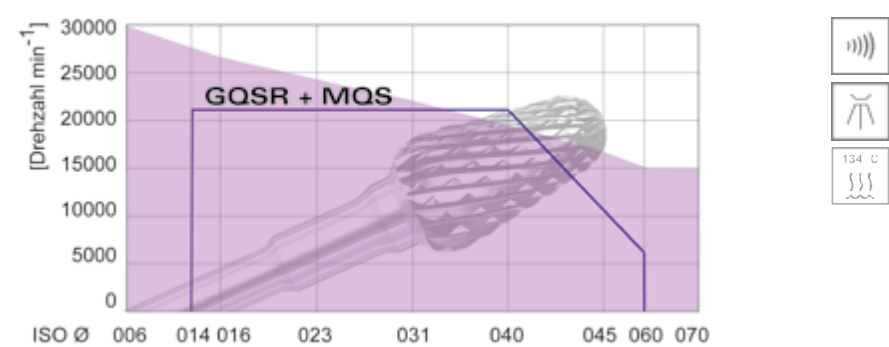
max. admissible speed

The mentioned maximum admissible speed (rpm) indicated on each BUSCH pack is not allowed to be exceeded due to safety regulations.

speed ranges

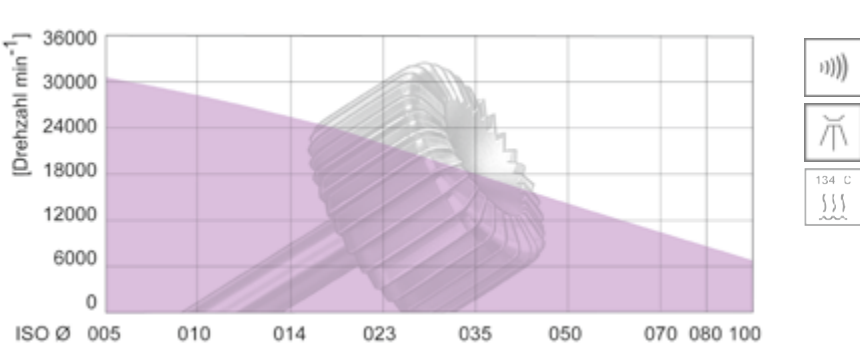
Keeping within the instrument-specific speed ranges leads to good treatment results (see diagram).

If the diameter of the working part exceeds that of the shank, powerful centrifugal forces may build up at higher speed which might bend the shank and/or fracture the instrument.


Rule of thumb: The larger the diameter of the instrument, the slower its speed.

Lower speeds should be selected in dry technique to keep any heat that might be generated to a minimum.

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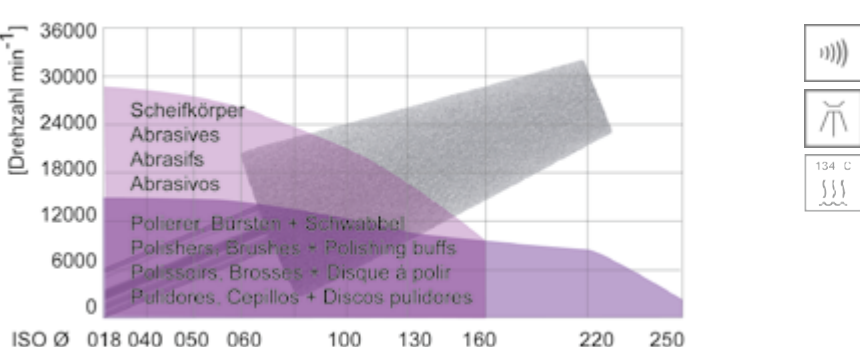The current reprocessing recommendations for rotating instruments can always be found at www.busch.eu.

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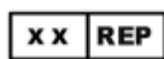
Modifications in the sense of progress are our tradition and therefore, reserved. Any reproduction, even in extracts, is subject to the written authorization of BUSCH & CO. GmbH & Co. KG. We wish to thank Mr Till Schwermer and Dr Norbert Scholz, MD for kindly allowing us to use the application photos.

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Our current catalogue
„Rotary instruments for
podology/footcare and
naildesign“ you can find
at www.busch.eu

Busch Quality Management
certified: DIN EN ISO 13485:2016
CE 0124

BUSCH & CO. GmbH
& Co. KG

Postfach 1152
D-51751 Engelskirchen

Unterkaltenbach 17-27
D-51766 Engelskirchen
GERMANY

Telefon: +49 (0) 2263 86-0
Telefax: +49 (0) 2263 20741
mail@busch.eu
www.busch.eu